
## MANUÁL PRO ELEKTRICKOU ODSÁVAČKU MLÉKA **COMFY** ZOP081100

- Ⓚ PRÍRUČKA PRE ELEKTRICKÚ ODSÁVAČKU MLIEKA COMFY
- Ⓔᅀ MANUAL FOR ELECTRIC BREAST PUMP COMFY
- Ⓓᅀ GEBRAUCHSANWEISUNG FÜR ELEKTRISCHE MILCHPUMPE COMFY
- Ⓕᅀ ELEKTROMOS MELLŐSZÍVŐ HASZNÁLATI ÚTMUTATÓ COMFY
- Ⓟᅀ LAKTATOR ELEKTRYCZNY – INSTRUKCJA OBSŁUGI COMFY
- Ⓘᅀ MANUALE PER IL TIRALATTE ELETTRICO COMFY
- Ⓛᅀ PRIROČNIK ELEKTRIČNEGA APARATA ZA SESANJE MLEKA COMFY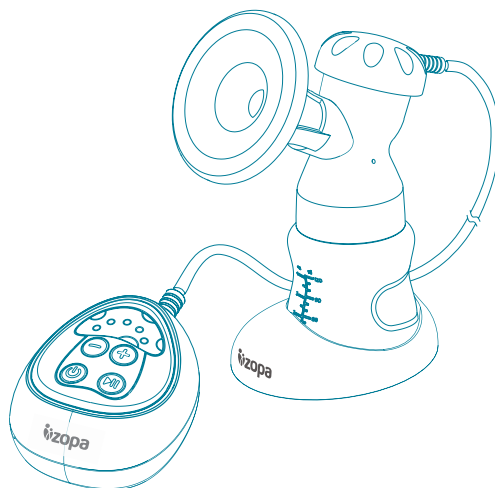

|            |           |
|------------|-----------|
| <u>CZ</u>  | <u>3</u>  |
| <u>SK</u>  | <u>8</u>  |
| <u>EN</u>  | <u>13</u> |
| <u>DE</u>  | <u>18</u> |
| <u>HU</u>  | <u>23</u> |
| <u>PL</u>  | <u>28</u> |
| <u>IT</u>  | <u>33</u> |
| <u>SLO</u> | <u>38</u> |

# MANUÁL K POUŽITÍ

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# USER'S MANUAL

## IMPORTANT! KEEP FOR FUTURE REFERENCE

### INSTRUCTIONS FOR ELECTRIC BREAST PUMP COMFY

- Unique soft silicon mimics the baby's sucking action and allows effective milk expression.
- USB connection allows use with all devices supporting USB.
- Compact size for easy portability.

### WARNING

1. Only for use by one person. Use by more than one person may present a health risk.
2. After use, always unplug the adapter.
3. When cleaning, ensure the adapter has been unplugged.
4. Do not clean the cap with water. If you find water in the cap, remove the membrane base from it and leave it to dry out before using it again.
5. Check that the tube is clean and there are no water droplets within it.
6. No components should be sterilised in a microwave oven or steriliser.
7. Do not subject the pump and motor unit to high temperatures or flames and keep out of the reach of children.
8. Heat and massage will reduce any pain which may occur while expressing milk.
9. Product design changes reserved.

## FUNCTIONS & PROPERTIES

### 1. Product functions


- (1) Microprocessor controlled suction mimics your baby's discontinuous sucking for maximum effectiveness.
- (2) In stimulation mode, you can choose 2 levels of suction force, which massage your nipples and promote milk creation and expression. In suction mode, there are 3 levels of suction, which mimic your baby's sucking rhythm.

### 2. Product properties

- (1) The membrane's unique design prevents milk backflow.
- (2) Soft silicon pads secure massage and comfortable expression.
- (3) Adapters for narrow-neck bottles included in pack.
- (4) USB cable allows for use with any product containing a USB port
- (5) Compact size for easy portability; LED diodes signalise suction force.

## TECHNICAL PARAMETERS

Power adapter  
 Power supply: 100V-240V ~ 50/60Hz  
 Output power: 6V 1A  
 Power 3.6W  
 Stimulation speed 100T/min  
 Suction speed 26-48T/min  
 Stimulation force: -0.010 ~ -0.018Mpa (max), 2 setting levels  
 Suction force: -0.20 ~ -0.036Mpa (max), 3 setting levels  
 Timer: 30 minutes  
 Noise: ≤60dB(A)  
 Operating conditions: 5-40°C, RH≤90%  
 Electric shock protection: Class II AC Adapter, Group III motor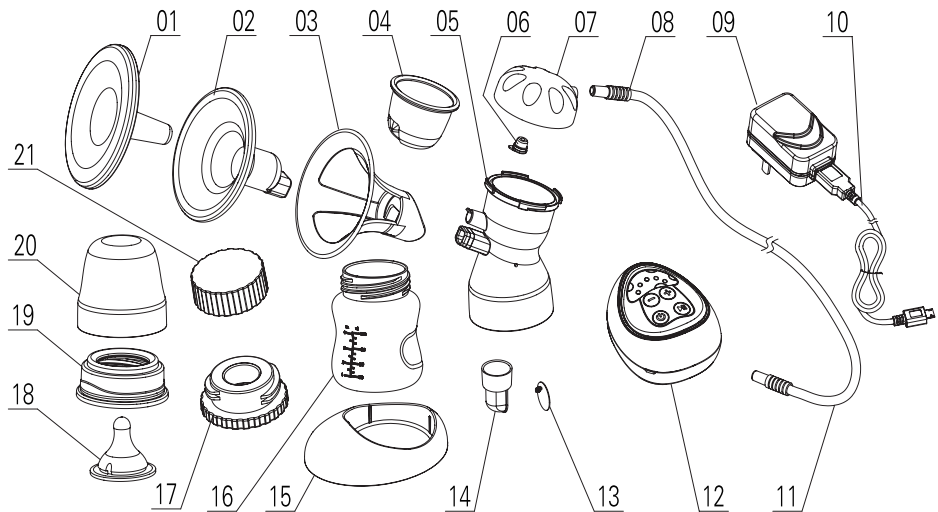

## PRODUCT DESCRIPTION

|                              |          |   |       |
|------------------------------|----------|---|-------|
| 01. Silicon breast pad guard | 1 PP     | 12. Motor unit                          | 1     |
| 02. Silicon breast pad       | 1 SIL    | 13. Vent membrane                       | 3 SIL |
| 03. Breast attachment        | 1 PP     | 14. Vent                                | 1 P P |
| 04. Membrane                 | 1 SIL    | 15. Bottle stand                        | 1 PP  |
| 05. Pump body                | 1 PP     | 16. 120ml bottle with wide neck         | 1 PP  |
| 06. Membrane base            | 1 SIL    | 17. Adapter for bottle with narrow neck | 1 PP  |
| 07. Cap                      | 1 PP     | 18. Teat                                | 1 SIL |
| 08. Tube connector           | 3 PP/TPE | 19. Securing collar                     | 1 PP  |
| 09. Power adapter            | 1        | 20. Bottle cap                          | 1 PP  |
| 10. USB cable                | 1        | 21. Bottle close cap                    | 1 PP  |
| 11. Tube                     | 2 SIL    |   |       |

## ASSEMBLY & CONNECTION

The following parts which come into contact with breast milk when used should always be cleaned and sterilised before every use: silicon breast pad, membrane, bottle attachment, vent membrane, vent, bottle, bottle close cap, adapter for bottle with narrow neck.

### 1. Breast pump assembly (breast attachment)

Fit the silicon breast pad (02) to the breast attachment (03) and affix to the pump body (05), then check that they are firmly attached.


### 2. Breast pump assembly (membrane)

Insert the membrane (04) into the pump body (05), affix the cap (07) and screw on firmly so no air can escape when you connect the tube (08).

### 3. Breast pump assembly (vent)

Attach the vent membrane (13) to the vent (14). The membrane (13) should stick to the vent (14) to prevent air from escaping. Now attach the vent (14) to the pump body (05). Check they are firmly attached so no air can escape.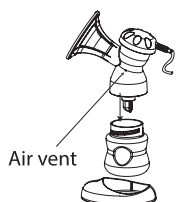



### 4. Breast pump assembly (bottle and bottle stand)

Screw the bottle (16) onto the pump body (05) and insert it into the bottle stand (15).

## CONNECTING THE BREAST PUMP TO THE MOTOR UNIT

### 1. Motor unit and breast pump

Connect the pump using the connectors on its ends to the pump body cap (07) and to the motor unit (12). The connection should be firm to prevent air from escaping.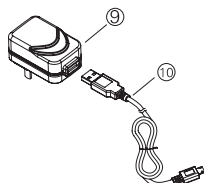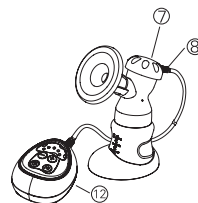


### 2. Adapter and USB Cable

Connect the USB cable (10) to the power adapter (09).

### 3. Motor unit and power adapter

Insert the micro-USB connector into the motor unit as displayed on the diagram and insert the power adapter into the socket. The LED diodes should light up and turn off one after another, showing it is on standby.

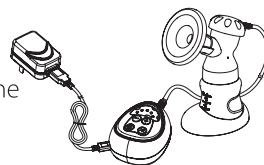
## USE

### 1. Before every use

- (1) Check the pump is clean and does not contain any liquid of any sort.
- (2) Clean, press and massage your breast, nipple and areola; relax to avoid pain when extracting milk.

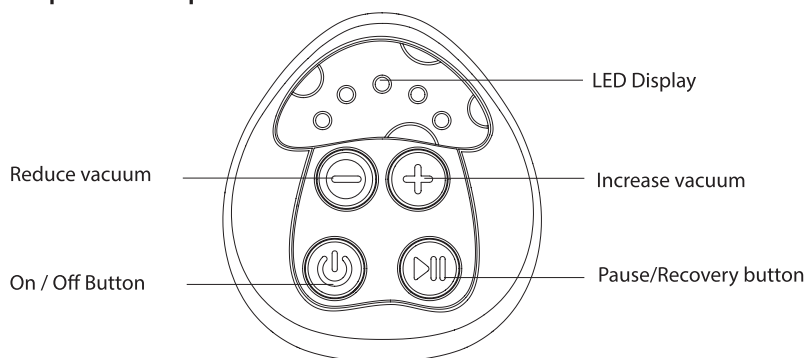### 2. Control panel description

### 3. Turning on/off

- (1) After plugging in the pump and pressing the on button,  the motor will run in stimulation mode at the lowest level for a period of 2 minutes, after which it will automatically move to suction mode. The timer is set for a total time of 30 minutes.
- (2) To turn off, press the off button  and the motor will switch to standby.

### 4. Setting suction force and speed

- (1) When turning on from standby, the pump will switch to stimulation mode for a period of 2 minutes, and then will move into suction mode at the lowest level (factory setting).
- (2) You can set the force of suction using the + and - buttons.
- (3) If you press the pause button  while pumping, pumping pauses, the LED diodes signalise the suction level set by flashing. If you want to continue pumping, press the pause button  again.

### 5 LED diodes

The LED diodes signalise the chosen programme and suction force.

## 6. Timer

- (1) When you press the on/off button  to start suction, the timer will be set to 30 minutes, and once this time is up the motor unit and LED diodes will switch off.
- (2) If you press the on/off button  while pumping, the motor unit will stop and the timer will reset. If you press the on/off button  again, the motor unit will start again and the timer will be set for 30 minutes again.

## PROBLEM SOLVING

### **The pump is not expressing milk, or is not expressing much.**

- Check all the connections are correct and firmly attached.
- Check the vent membrane is correctly pressing on the vent.
- Check the membrane base is correctly fitted.
- Check the tube is not bent or broken and there is no water inside it.

### **The motor unit isn't switching on**

- Check the power adapter is plugged in to a socket, or the USB connector is connected to a USB port and that it is connected to the motor unit.

### **There is milk backflow**

- If you find milk or water in the membrane or in the tube, clean the tube with clean hot water. Dry out thoroughly before using it again. Check the membrane has not been damaged.

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